 ITSETESTI SYPHILISTARTUNNAN (KUPPA) SELVITTÄMISEEN  
 SELF-TEST FOR DETERMINING SYPHILIS INFECTION

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## INSTRUCTIONS FOR USE

Thank you for purchasing Medigoo.com quick health test kit. This test kit is used for the qualitative detection of Syphilis (Treponema Pallidum antibody) in blood.

Syphilis is a venereal disease that progresses through distinct stages of infection: primary, secondary, tertiary and quaternary. These stages produce diverse clinical symptoms, typically producing initial sores known as chancres then syphilitic rash followed by long periods of dormancy. Untreated infection eventually leads to cardiovascular problems and neurosyphilis. Caused by spirochete Treponema pallidum (TP), the infection is usually acquired by sexual contact, although the disease may be transmitted by transfusion of infected blood. Intrauterine infection also occurs. The organism has proven virtually impossible to cultivate, and diagnosis of the infection usually depends on the detection of antibodies in the blood, which appear soon after initial infection.

## PRODUCT CONTENTS

**The box contains all necessary items to perform a test:**

- 1 Sealed aluminium pouch containing:
  - 1 test device, 1 plastic pipette, 1 desiccant pouch
- 1 diluent bottle containing 1 mL of diluent
- 2 sterile lancet for blood sampling
- 1 instruction leaflet.

Open the protective aluminium pouch only when you are ready to perform the test. The desiccant bag should not be used.

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## PRECAUTIONS

1. This test is exclusively intended for in vitro diagnostic only. DO NOT SWALLOW.
2. Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully followed. Follow strictly the indicated blood and diluent quantities, as well as reading times.
3. Store between +4 °C and +30° C. Do not freeze.
4. Do not use after the expiry date printed on the label and on the protective pouch or if this pouch is damaged.
5. Do not reuse test.
6. Keep out of reach of children.
7. After use, all the components can be discarded along with the household garbage disposal.

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## QUESTIONS AND ANSWERS

### How does the Syphilis test work?

When present in the body, TP induce the production of specific antibodies by the immune system. These circulating antibodies are able to recognize the spirochaete and to stick on them. Syphilis test detects specifically these anti-TP antibodies in blood and therefore shows the spirochaete presence.

### When can this test be used?

Syphilis test should be performed in case of appearance of clinical symptoms, the main ones being chancres and specific rash. The test can be performed at anytime of the day.

### Can the results be incorrect?

The results are accurate as far as the instructions are carefully followed. Nevertheless, the result can be incorrect if Syphilis test gets wet before taking the test or if the quantity of blood added to the sample well is not correct. The pipette provided in the box allows making sure the collected blood volume is correct. Very early stage of the infection could also lead to negative results due to the low concentration of anti TP antibodies in blood.

### How to interpret the test if the colour and the intensity of the lines are different?

The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as positive whatever the colour intensity of the test lines.

### What is the line that appears at the mark C (Control) ?

When this line appears, it only means that the test was performing well.

### If I read the result after 15 minutes, will the result be reliable?

No. The test should be read within 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

### What do I have to do if the result is positive?

If the result is positive, it means that anti-TP antibodies are present in the blood and that you should consult a doctor to show the results. Then, the doctor will decide whether additional examination should be performed.

### What do I have to do if the result is negative?

If the result is negative, it means that the test could not detect anti-TP antibodies in the blood sample. Meanwhile it is recommended to consult a doctor if symptoms persist.

### What is the accuracy of the Syphilis test?

The test is accurate and has been used for more than 15 years by professionals in the field. Although this test is reliable, false positive or false negative results can be obtained. Evaluation reports show an overall agreement of at least 97.6% with reference methods.

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🇬🇧 Wash your hands thoroughly. Use soap and warm water. Dry your hands with a clean towel.

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🇬🇧 Prepare the lancet. Twist off and discard the lancet's grey cap without touching the trigger button.

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🇬🇧 Massage the hand without touching the future puncture site by rubbing down the hand towards

the fingertip of the middle or ring finger. Choose the finger you use the least. You can clean the fingertip by a wipe damped in disinfectant or alcohol.

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🇬🇧 Strongly press the tip of the lancet towards previously chosen finger. Puncture the skin by pressing the trigger button. The tip retracts automatically and safely after use.

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🇬🇧 Gently rub the hand from the wrist and palm down to the finger to form a rounded drop of blood over the puncture site.

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🇬🇧 Without pressing the pipette bulb, put it in contact with the blood drop. The blood migrates into the pipette through the capillary effect to the line indicated on the pipette. You may massage again your finger to obtain more blood if the line is not reached. Take blood only upto the line on the pipette. Avoid air bubbles.

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🇬🇧 Put all the collected blood from the pipette into the round sample well (s) of the cassette, by pressing on the pipette bulb.

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🇬🇧 Wait 30-40 seconds for the blood to be totally absorbed into the sample well (s). Unscrew the blue cap of the plastic dropper bottle (while keeping the white cap tightly screwed in), turn the bottle upside down and add exactly 4 drops of diluent into the sample well (s) with an interval of 2-3 seconds between each drop.

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🇬🇧 Read the results at 10 min. Do not interpret the results after 15 min.

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Only one coloured line appears at the mark C (Control). This result means that no anti-TP antibodies are detected in the blood sample. No syphilis infection detected.

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Two coloured lines appear in the window under the marks T (test) and C (control). The intensity of the line T may be more transparent than the intensity of the line C. The result means that the blood sample contains anti-TP antibodies and that you should consult a doctor. Syphilis infection detected.

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Test failed: One red line is shown in the Test area (T) and no line in the control area C. Alternatively, no lines are shown either at test area T or the control area C. Repeat the test with a new testing device and with fresh blood.

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©Medigoo 2016-2018
Owen Mumford Ltd