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 **INSTRUCTIONS FOR USE**

Thank you for choosing Medigoo.com quick health test kit. This health test kit is intended for the early detection of anti borrelia IgM antibodies to diagnose borrelia bacterium. Borrelia bacteria can cause Borreliosis (Lyme disease) after 3-30 days of the tick bite and symptoms may be visible already that time. It is recommended that this test is made 2-4 weeks after the bite to ensure the detection of the antibodies from the blood sample.

Ticks are insects living in undergrowth and forests in temperate climatic zones. Around half of the ticks are infected by a bacterium (Borrelia) that can be then be transmitted to humans with a few percent transmission risk if the tick is attached less than 4 hours on the arm, the leg and any other part of the body. Borrelia bacterium affects the nervous system and can lead to serious neurological disorders. European strains of Borrelia include B. afzelli, B. garinii, B. burgdoferi sensu stricto, B. spielmanii, and B. bavariensis which are known to be pathogenic. Species B. valaisiana and B. lusitaniae are potentially pathogenic. That infection can also cause cutaneous lesions and arthritis. From 3 to 10 days, or even several weeks, after being bitten, an inflammation and redness (erythema) can appear in the bite region (50% of patients). The inflammation can be accompanied by feverishness. In case of infection, the immune system reacts by releasing specific antibodies directed against Borrelia bacterium. A particular category of antibodies (IgM: immunoglobulins M) which appears first can be detected between 2 and 6 weeks after being infected. Borreliosis (Lyme disease) develops in 3 stages: In phase I (2 to 4 weeks), the IgM antibodies appear only in 40 to 60 % of the cases. During the phase II (4 to 6 weeks later), the antibodies are detectable in 70 to 90 % of the cases. At last during phase III, the antibodies are generally present and can be detected.

**PRODUCT CONTENTS**

**The box contains all necessary items to perform a test:**

- 1 Sealed aluminium pouch containing:
- 1 test device, 1 plastic pipette, 1 desiccant pouch
- 1 diluent bottle containing 1 mL of diluent
- 2 sterile lancets for blood sampling
- 1 instruction leaflet.

Open the protective aluminium pouch only when you are ready to perform the test. The desiccant bag should not be used.

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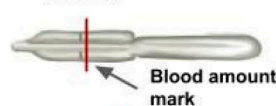
**PRECAUTIONS**

1. This test is exclusively intended for in vitro diagnostic only. DO NOT SWALLOW.
2. Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully followed. Follow strictly the indicated blood and diluent quantities, as well as reading times.
3. Store between +4° C and +30° C. Do not freeze.
4. Do not use after the expiry date printed on the label and on the protective pouch or if this pouch is damaged.
5. Do not reuse test.
6. Keep out of reach of children.
7. After use, all the components can be discarded along with the household garbage disposal.

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🇬🇧 Wash your hands thoroughly. Use soap and warm water. Dry your hands with a clean towel.

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🇬🇧 Open the foil pouch (from the notch) and only use the test cassette and the pipette. Dispose of the desiccant pouch.

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🇬🇧 Prepare the lancet. Twist off and discard the lancet's grey cap without touching the trigger button.

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🇬🇧 Massage the hand without touching the future puncture site by rubbing down the hand towards

the fingertip of the middle or ring finger. Choose the finger you use the least. You can clean the fingertip by a wipe damped in disinfectant or alcohol.

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🇬🇧 Strongly press the tip of the lancet towards previously chosen finger. Puncture the skin by pressing the trigger button. The tip retracts automatically and safely after use.

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🇬🇧 Gently rub the hand from the wrist and palm down to the finger to form a rounded drop of blood over the puncture site.

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🇬🇧 Without pressing the pipette bulb, put it in contact with the blood drop. The blood migrates into the pipette through the capillary effect to the line indicated on the pipette. You may massage again your finger to obtain more blood if the line is not reached. Take blood only upto the line on the pipette. Avoid air bubbles.

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🇬🇧 Put all the collected blood from the pipette into the round sample well (👉) of the cassette, by pressing on the pipette bulb.

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🇬🇧 Wait 30-40 seconds for the blood to be totally absorbed into the sample well (👉). Unscrew the blue cap of the plastic dropper bottle (while keeping the white cap tightly screwed in), turn the bottle upside down and add exactly 4 drops of diluent into the sample well (👉) with an interval of 2-3 seconds between each drop.

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🇬🇧 Read the results at 10 min. Do not interpret the results after 15 min.

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## QUESTIONS AND ANSWERS

### How does Borreliosis test work?

In presence of Borrelia bacterium transmitted by a tick, specific IgM antibodies are produced first by the immune system. The antibodies circulate in the blood and bind on the encountered bacteria. The Borrelia antigens coated on the borreliosis test strip specifically detect those antibodies by producing a coloured line under the T mark of the cassette. A control line capturing the reagent excess appears as a coloured line under the C mark of the cassette. In case both lines appear, the presence of Borrelia bacterium can be suspected.

### When should the test be used?

Borreliosis test should be performed between 2 and 6 weeks after being bitten by a tick or as soon as a round inflammation (erythema) appears on a part of the body likely which was exposed to a tick bite. It corresponds to the phase 1 of the disease for which the borreliosis test is well suitable and for which the test sensitivity is 53,8%. Contamination happens only with a few percent probability, but the positive result and the presence of IgM antibodies indicates an infection that should be rapidly treated.

### Can the result be incorrect?

The results are accurate as far as the instructions are carefully followed. Nevertheless, the result can be incorrect if Borreliosis test gets wet before taking the test, or if the quantity of blood dispensed in the sample well is not sufficient. The plastic pipette provided in the box allows making sure the collected blood volume is correct.

### How to interpret the test if the colour and the intensity of the lines are different?

The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as positive whatever the colour intensity of the test line is.

### What is the line that appears under the mark C (Control) for?

When this line appears, it only means that the test was performing well.

### If I read the result after 15 minutes, will the result be reliable?

No. The result should be read at 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

### What do I have to do if the result is positive?

If the result is positive, it means that IgM antibodies directed against Borrelia bacterium are detectable in blood and that you must absolutely consult a doctor showing him the test result. Then, the doctor will order a treatment to follow in order to avoid complications arising from this infection. In case of past infection having occurred several years before, a positive result can be obtained as the antibodies may be remaining in some persons. A positive result can also be obtained in patient having been efficiently cured with antibiotics.

### What do I have to do if the result is negative?

If the result is negative, it means that no IgM antibodies directed against Borrelia bacterium are detectable in blood. **Caution:** Falsely negative results can be observed during phase I (antibodies appearing in 40 to 60% of the cases) or during phase II (antibodies rising in 70 to 90% of the cases) of the disease development. In case of negative result it is strongly recommended to perform a new test 2 to 6 weeks later in order to check the increase of antibodies level. If you have been preventively treated by antibiotics, the antibodies are not detectable or cannot be detected. If the tick's bite is too recent (less than 2 weeks), your organism has not had enough time to produce detectable antibodies. It is recommended to repeat the test with a new borreliosis test between 2 to 6 weeks after the presumed bite. However, if a rash, accompanied or not by feverishness, persists, it is recommended to consult your doctor.

### What is the accuracy of Borreliosis test?

The Borreliosis test is accurate and is used by professionals in the field (hospitals, laboratories...). A study performed on a panel of 251 patients samples shows an overall result correlation of 88% when compared to reference methods. However, it is still possible that the IgM antibodies appearing after an infection by Borrelia are not detected due to late maturation of the immune system of certain persons depending on the level of the disease. Despite the reliability of the test, falsely positive or falsely negative results are possible. This is especially true for patients being seropositive for the cytomegalovirus (CMV) which may show false positive results when using the borreliosis test.

Information on Borrelia infections and related disease

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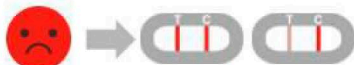
Only one coloured line appears at the mark C (Control). This result means that antibodies directed against Borrelia bacteria were not detectable in blood.

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Two coloured lines appear in the window at the marks T (test) and C (control). The intensity of the line T may be more transparent than the intensity of the line C. This result means that IgM antibodies directed against Borrelia bacterium are present in blood and an infection is starting. You must absolutely consult your doctor.

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Test failed: One red line is shown in the Test area (T) and no line in the control area C. Alternatively, no lines shown either at test area T or control area C. Repeat the test with a new testing device and with fresh blood.

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©Medigoo 2016-2018
**Owen Mumford Ltd**