Virtsatietulehdustesti

Urinvägsinfektion Test

Urinary tract infection test

©Medigoo 2016-2018

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INSTRUCTIONS FOR USE

Thank you for purchasing Medigoo.com personal health quick test kit. This Urinary Tract Infection (UTI) test is for determination of possible urinary tract infection from an urine sample. The Urinary Tract Infection (UTI) Urine Test Sticks contains solid phase reagent areas affixed to a plastic stick. They are provided as a dry reagent. The Urinary Tract Infection (UTI) Urine Test Sticks provide tests for the semi-quantitative determinations of nitrite and leukocytes in human urine.

PRODUCT CONTENTS

1pc Urine test stick (2-parameters: Nitrite and Leukocytes)

Color label chart attached to the test stick

Instructions for use

Materials required but not provided:
Clock or timer

IMPORTANT NOTICE: YOU MUST READ THIS INSTRUCTION MANUAL BEFORE RUNNING THE TEST. IF YOU HAVE ANY POSITIVE RESULTS OR NEGATIVE RESULTS BUT THE SYMPTOMS CONTINUE, YOU SHOULD CONSULT YOUR DOCTOR. Similar symptoms to urine tract infection symptoms are caused by an STD (Sexually Transmitted Disease) as well.

PRECAUTIONS

- For in Vitro diagnostic use.
- Do not touch test areas of the test stick.
- Working area should be free of detergents and other contaminants.
- Store at room temperature, out of direct sunlight. Do not freeze.
- Do not use after expiration date.
- Store all test sticks in sealed a pouch. Do not remove the desiccant from pouch.

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Wash your hands with soap and warm water or clean with an alcohol swab. Allow to dry.

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Remove a 2-parameter (Nitrite and Leukocytes) UTI test Stick from the pouch.

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Hold the absorbent tip in urine for at least 10 seconds. Make sure both test strips are wetted with urine. You can use also a plastic cup for urine collection and dip the test strips into urine for 10 secs.

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Wait for 2 min to read the result. Compare the test results carefully with the color chart in good light. Changes in color that appear only on the edges of the test areas or after more than 3 minutes are of no clinical significance, and should not be interpreted.

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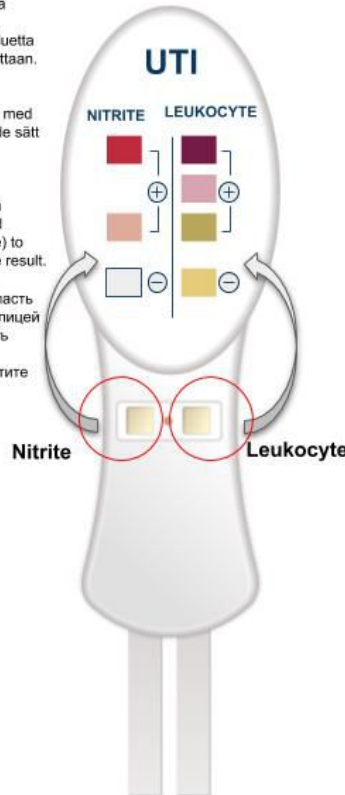
Compare nitrite test area (nitrite) to nitrite test chart and leukocyte test area (leukocyte) to leukocyte test chart. Read the result.

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NEGATIVE RESULT: **NITRITE** **LEUKOCYTE**

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NEGATIVE RESULT WHEN BOTH NITRITE AND LEUKOCYTES ARE NEGATIVE. No color change can be visually seen or the leukocytes test area changes to slightly yellow. NO URINARY TRACT INFECTION DETECTED.

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POSITIVE RESULT: **NITRITE** **LEUKOCYTE** **NITRITE** **LEUKOCYTE** **NITRITE** **LEUKOCYTE**

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IF EITHER OR BOTH OF THE NITRITE AND LEUKOCYTES ARE POSITIVE THE RESULT IS POSITIVE. **Nitrite:** Any degree of pink color after 30 seconds indicates a positive test for nitrite suggesting a clinically significant infection by bacteria. Pink spots or pink edges should not be interpreted as a positive result. **Leukocyte:** Dark or light purple color in leukocytes indicates a positive result. A negative result does not necessarily rule out bacterial infection. URINARY TRACT INFECTION.

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